


AUTOMATIC HAND SANITIZER DISPENSER

Product manual

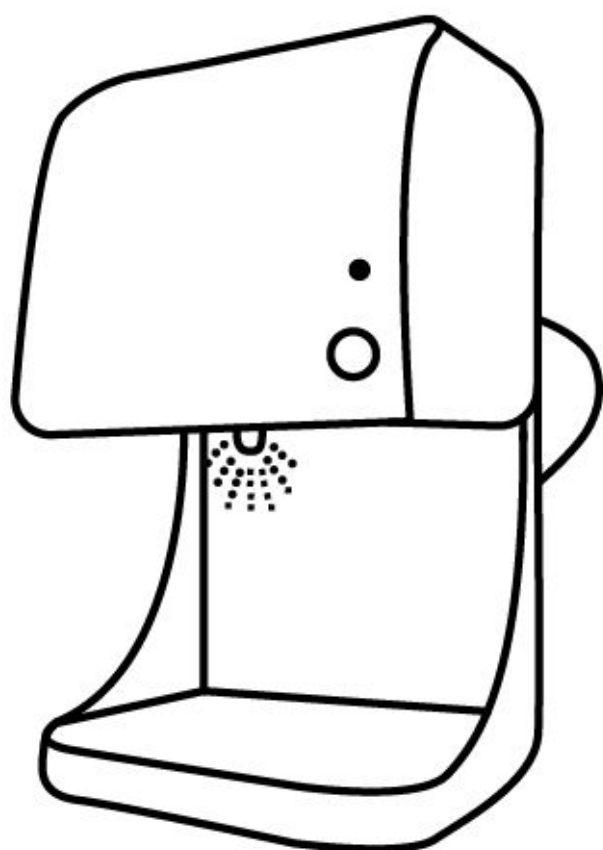

**Before using this product, please reading this handbook
carefully and keep it well for consultation**

Table of contents

1. Matters needing attention	Page 1
2. Major technique parameter	Page 1
3. Appearance and parts name	Page 2
4. Attachment	Page 3
5. Product setting	Page 3
6. Operation method	Page 3
7. Method of pouring the disinfectant	Page 5
8. How to clean the vessel	Page 5
9. How to clean the nozzle	Page 6
10. Routine maintenance	Page 6
11. Common breakdown and servicing	Page 7
12. Product Maintenance	Page 8
13. How to choose the hand disinfectant	Page 9

[Respectful user]

Thanks for your choosing the brand of-Automatic Hand&Arm Sanitrzer.

·Before using the product,you'd best read the "Product User's manual" carefully in order to handle our product in a proper way.

·Taking notice to "Matters needing attention","Operating method","How to choose the hand disinfectant",and please keeping the handbook well for consultation.

Notice


- Power voltage :AC 220~240V±10%, 50/60Hz.
- It is danger to tear down the machine and snip the electrical wire by yourself.
- The machine should be installed in the clean place and be kept away form humidity,dust,sunlight,and fire.
- Keep the machine clean, especially the area of inductor.
- Choose the proper disinfectant and do not use viscous liquid or liquid that does harm to skin.
- In order to avoid spilling ,you should use the special funnel when pouring disinfectant and keep the funnel far away from children.
- To keep the machine in the condition of good work disinfectant must be supplied in time.Avoiding the machine empty.
- If not using for a long time, pour the residual liquid, pack it tight and place it in a clean position Do not invert.

[Main Technial parameters]

Power voltage	AC220~240V± 10%,50/60Hz
Input power	≤25W
Safety standard	I series Btype
Spray type	Continuous spray
	Hand sensing
Induction distance	8 ~13cm
Environment temperature	5°~ 40℃
Environment humidity	≤80%
Liquid amount	Full: 2000±10ml
	Empty: 100±10ml

Maximum capacity	2000ml
Fuse	5A
Storing	Normal room temperature
Operation standard	GB9706 Q/DXJRT013-2009
Net weight	3.3Kg
Gross weight	4.0Kg
Hole diameter	Ø6mm
Hole space between	130mm
Size	220x185x380mm
	The product will be developed with circumstance. Any change will not be informed spectally.

[Appearance and parts name]

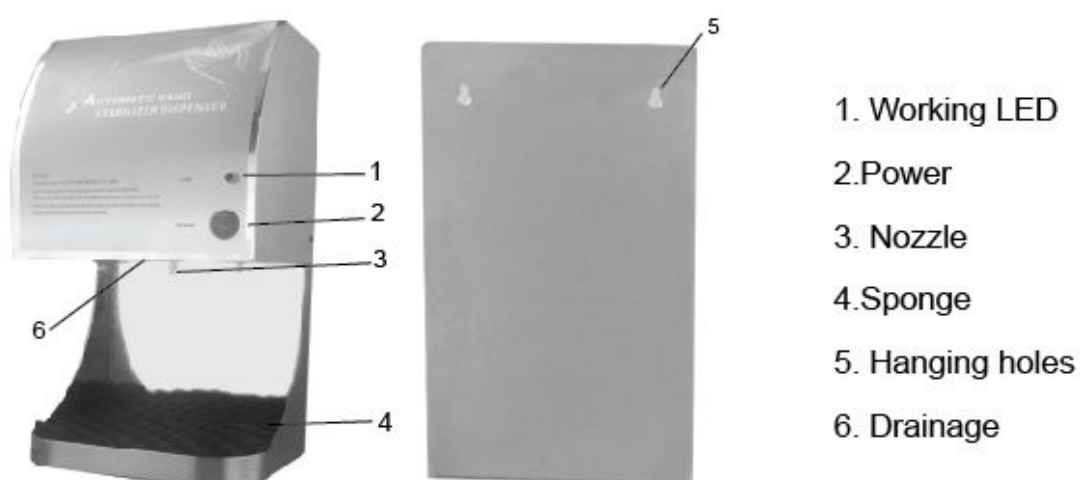

[Attachment]

Attachment	Quantity
Manual	1 set
Warranty	1 sheel
Maintenance Card	1 sheel
Fuse 5A(Ø5x20mm)	1 piece
Expansion tube	4 piece
(Ø8)Screws	2 piece
Funnel	1 piece

[Product setting]

Double-hole setting

- 1.To make two holes in the wall with a drill. The distance between holes and floor is about 145cm~160cm.
- 2.Hole diameter isØ8mm. Holes space between is 130mm (the two holes must be horizontal).
- 3.After putting the expansion tubes into each hole and tightening the screws, then hang the machine on the wall.

[Operation method]

[1] Preparation

Step: Before using the sanitizer, you must clean the disinfectant vessel as the following steps.

1. when the first time using, please to be working last for 10 seconds to be drained the air which in the pipe line.
- 2 . Pour 75% alcohol in the vessel and keep it in for 30 minutes, then clean the vessel.
3. When you pour the liquid into vessel, you must use the funnel with filter paper to avoid the nozzle jam.
4. You must pour the liquid into the vessel carefully and be sure that the liquid is not overflowed. The liquid level must be under the red warning line.
5. You must tighten the cover in the end.
6. Adjusting the Spray amount adjusting knob to rightmost side, hung the sterilizer and turn on the switch.
7. Extending hands to the induction area for several times, get rid of all of the air in the tube. Then you can use the sterilizer in normal way.

Illustration



▪ Pour the liquid with funnel

[2] Operation

Step: Turn on the switch (green button)

1. When you extend the hands to the induction area, the sanitizer spray liquid.
2. After spraying liquid, the sanitizer will come back to be in the standby status.
3. When the capacity of the disinfectant is under 100ml, the liquid LED is flashing, you should add the disinfectant in time.

[Method of pouring the disinfectant]

Steps

1. When adding the disinfectant, you should use the special funnel.
2. You must pour the liquid into the vessel carefully and be sure that the liquid is not overflowed.
3. When the disinfectant is full, the buzzer will alarm, the liquid is full.
4. You must tighten the rubber cover and push the slide cover on the top of the machine.

[How to clean the vessel]

1. The disinfectant vessel should be cleaned and sterilized every week.
2. Switch off the power and open the drainage.
3. Put vessel in 75% alcohol and wash it completely.

[How to clean the nozzle]

1. Unscrew the nozzle from the machine first and then unscrew the nozzle to two parts.
2. Put the nozzle in alcohol or in the boil water to get rid of the impurity.
3. After cleaning screw the nozzle to a whole and screw the nozzle to the machine
(Must tighten the nozzle,otherwise the spray effect should be influenced).

Illustration



■ Nozzle

[Routine maintenance]

Part	Content
Switch off power	You should switch off power before cleaning the sanitizer.
Clean the shell	Please clean the shell of sterilizer with the dry cloth. Please not use volatile liquid to wipe the sterilizer such as: thinner, benzene, and gas.
Clean the sponge	wash out sponge.
Working environment	The machine should be installed in the clean place and be kept away from humidity, dust, sunlight, and fire.
Cleaning the vessel	The vessel should be cleaned every week. Unscrew the drain,using the funnel, pouring the 75% alcohol washing several times.
Clean the funnel	Please clean the funnel with soft wash.

Product storing	If not using for a long time, switch off the power and pour the residual liquid.
	Add clean water in the vessel, and sway the vessel to wash the inside of it completely. Unscrew the drain port to discharge the water completely.
	Firstly, add the 75% alcohol in the vessel and keep the alcohol in the vessel for about 30 minutes. And then pour the alcohol and cover the lid. Finally, pack it tight and place it in a clean position.

[Common breakdown and servicing]

Malfunction	Causation	Obviation
The work indicating lamp is not bright	The power switch has not put through	Turn on the switch
	The power supply is not connected	Check-up the connection
	The fuse breaks	Check-up the fuse
The sanitizer can not work	Hand is too far away from the induction area	Extends the hand from the induction within 8-13cm
	Induction window is dirty	Clean the induction window
Nozzle jam	There is impurity in the liquid	Use the funnel to add liquid
		Use proper liquid
	Not using for a long time, impurity jam the nozzle.	Put the nozzle in alcohol or in the boil water to get rid of the impurity.
	If the working environment is not clean, the nozzle is very easy to be jammed by the impurity.	Unscrew the nozzle to clean it or change the nozzle.

Liquid dripping	Liquid overflow	The buzzer alarm the liquid is full
The spray amount is not enough	The network voltage is extremely low	The sterilizer can not be used when the voltage is low. You can only use it when the power voltage is AC220~240V \pm 10%.
<p style="text-align: center;">Explanation</p> <p>1 .If you can not obviate the malfunction yet after trying the suggested methods, please connect with the franchiser to get further help.</p> <p>2.You should not to teardown the sterilizer by yourself, otherwise, you will lose the maintenance qualifications.</p>		

[Product maintenance]

Guarantee terms

<p>1. Guarantee period is 12 months. Guarantee services will be provided only if the invoice manifold or sales receipt (indicating the date of purchase), is presented with the defective product within the guarantee period. Our company may refuse free-of-charge guarantee service if these documents are not presented or if they are incomplete or illegible.</p>
<p>2. When any quality problem being confirmed in 30 days after sale, we will offer the free change of the product basing on the invoice manifold (not offer the new package), and the highway freight charge (the fee exceed the normal freight standard should be paid by client).</p>
<p>3. To surpasses the warranty period, not to have the product which "Product Maintenance Card", the purchase invoice loses. Our company provides the paid service, only collects the materials.For the product beyond the guarantee period or without guarantee card, we can offer the maintenance, just receiving the cost, and not paying for the freight charge.</p>

4. This guarantee does not cover.

- Repair or attempted repair by persons who are not our company member.
- Damage or defects caused by use, operation or treatment of the product inconsistent with normal personal or domestic use.
- The model name or serial number on the product has been altered, deleted, removed or made illegible.
- Uncertain natural disaster accident.

[How to choose the hand disinfectant]

- Using the 75% alcohol is the best choice of the sanitizer.
- Hand disinfectant is not harm to the skin. We feel not uncomfortable when we use it for long time.
- Do not use the liquid which could cause allergic response or harmful irritation to skin, the liquid with abnormal smell, or the viscous liquid.

— Notice —

Before use the product, you'd best read the "Product manual" carefully in order to handle our product in a proper way.

Taking notice to "Matters needing attention ", "Operation method" , " How to choose the hand disinfectant", and please keeping the handbook well for consultation.